



# SARS-CoV-2 Antigen Rapid Test

## Product Features


- High Accuracy, Specificity and Sensitivity
- Test results within 15-30 minutes
- 12 Months Shelf Life
- Room Temperature Storage
- Oropharyngeal Swab and Nasopharyngeal Swab Available
- Control Set Available

## Clinical Performance


Relative Sensitivity: 92.30%

Relative Specificity: 99.40%

Accuracy: 98.98%

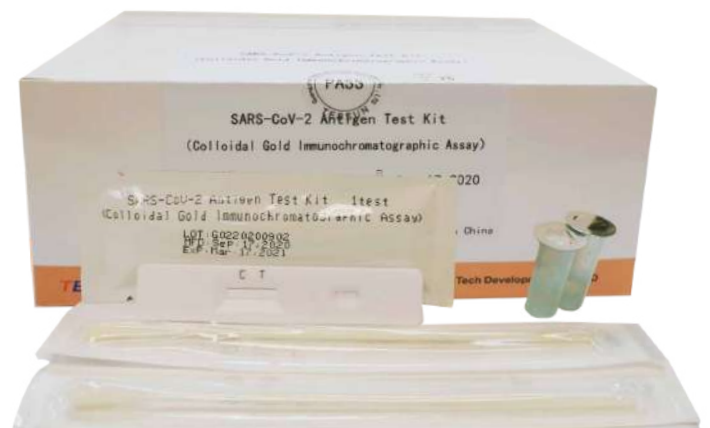

# Sampling

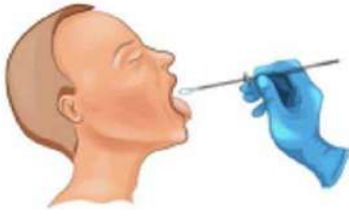

Oropharyngeal Swab




Nasopharyngeal Swab

## Test Procedure

1. Rip the aluminum foil pouch to take out the cassette, and put it on a clean place.
2. Specimen collection: according to standard operating methods.
3. Extraction specimen: insert the swab into an extraction reagent tube. Squeeze and stir the swab for more than 5 times.
4. Squeezing the sides of the tube when remove the swab to extract the liquid from the swab.
5. Press the nozzle cap tightly into the tube, and add 2 drops of extracted specimen to the specimen well of the cassette.
6. Observe the results in 15 minutes. The test is invalid clinically after 30 minutes.

## Interpretation of Test Results

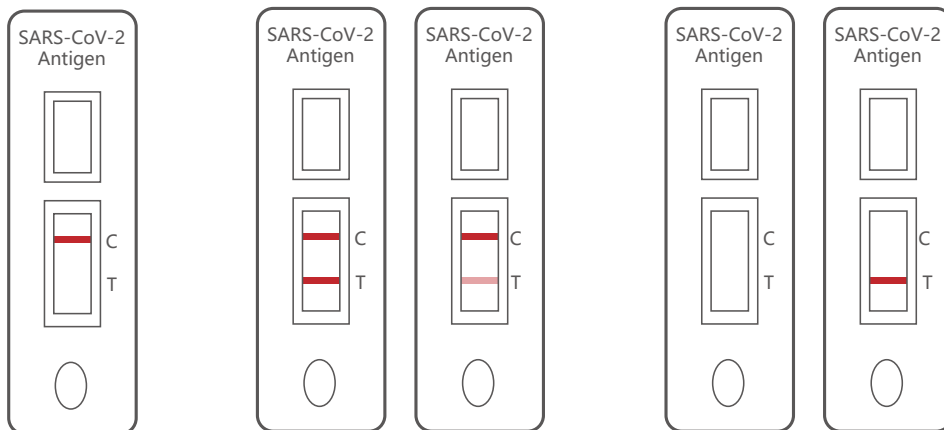

Negative

Positive

Invalid

Guangzhou Tebsun Bio-tech Development Co., Ltd

Address Room 2415,24/F,CITIC Plaza, 233 Tianhe North Road,Tianhe District,Guangzhou

Tell:400-888-4840

Websit:www.tebsun.com



TEBSUN  
天宝颂原

